# normo mar<sup>®</sup> spray

## Agua de mar ISOTÓNICA

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### ISOTONIC

**Strong force**  
**DAILY NASAL HYGIENE**

#### **Normomar Isotonic Sea Water Strong force** **Nasal spray**

A proper nasal hygiene is important for children and adults. The excess of the mucus in the nasal cavities may increase the microbial proliferation and the excessive dryness of the nasal mucosa.

**Normomar Isotonic Sea Water Strong force Nasal spray** hydrates strongly nasal cavities thanks to its mechanical action and helps washing and extracting the mucus, which easily allows the permeability.

**Normomar Isotonic Sea Water Strong force Nasal spray** maintain the specificity of the vital properties of natural sea water.

**Normomar Isotonic Sea Water Strong force Nasal spray** comes from the Bay of Cancale in Brittany, renowned for the quality of its water. The Bay of Cancale boasts an exceptional tidal range (12 m) and is swept by strong prevailing currents preventing pollution of the waters. This unique combination ensures that the water constantly clean itself and is therefore consistently pure. Sampling is performed automatically when the water quality is optimal (after the high tide period). Sampling area is classified as Class A and is under continuous health monitoring controls (microbiological, chemical contamination, phytoplankton, pH...).

#### **Composition:**

Purified water and sea water (sodium chloride 0.9%).

#### **Indications:**

Proper use of **Normomar Isotonic Sea Water Strong force - Nasal spray** in daily nasal hygiene of children and adults allows a therapeutic aid in some basic indications such as: nasopharyngitis, obstructive rhinitis and sinusitis, dry nose, hoarseness due to dry mucous membranes, adjuvant in ORL and pediatric treatment.

#### **Directions for use:**

1. Remove the protective cap.
  2. Introduce gently the nasal applicator in one nostril and press the pump to apply the product.
  3. Let drain the excess solution in order to extract the mucus, and then blow.
  4. Repeat the process with the other nostril.
  5. Clean the tip with water and dry.
  6. Cover the nasal applicator with the protective cap until the next use.
- Apply in each nostril as often as necessary.

#### **Contraindications and adverse effects:**

- No adverse effects have been described at the recommended therapeutic doses, but if you experience side effects, consult your usual doctor or pharmacist.
- Any serious adverse event related to this product must be declared to the manufacturer, distributor and national health authority in your country.

#### **Warnings and precautions:**

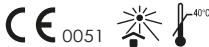
- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.
- Keep out of reach of children.

#### **Conservation and expiry date:**

- Do not use the product after the expiry date shown on the box and the bottle.
- Maintain in a dry and cool place.
- Deposit the bottle and the protective cap in the corresponding containers.

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**LABORATORIOS NORMON, S.A.**